

# New Depend Bariatric Strap for Briefs

Providing comfort and protection for all residents



- Straps attach to each side of briefs to extend the size and ensure a secure fit around a larger waist
- 20 pairs of straps per pack, 4 per pack
- \$1.40 per pair
- Available 1<sup>st</sup> of June 2019
- Code 17401



# Bariatric Strap Fitting Instructions



With resident on side gently fit DEPEND® Brief into groin with tape section at the back.



Expand the back of the DEPEND® Brief and smooth before rolling resident onto back.



Expand the front of DEPEND® Brief and smooth.



Fasten the strap to the middle of the side flap on the brief with the grip tab. Repeat this step for the opposite side flap.



Bring the strap from the back section over the front and fasten the tab to the front of the brief. Repeat this step for the other strap.

