




Quicktrach II

Cricothyrotomy is an emergency procedure to avoid death from suffocation in case of upper airway obstruction if intubation or tracheotomy are impossible.

- Sterile sets are pre-assembled and immediately ready for use
- Specially grinded needle tip makes prior incision via scalpel unnecessary and reduces the risk of bleeding
- Stopper prevents needle from being inserted too deep to reduce the risk of posterior tracheal wall perforation
- Necktape allows a fast and safe fixation
- Quickest airway access via one-step procedure
- Thin cuff seals the trachea and allows efficient ventilation with aspiration protection
- Stopper and safety clip reduce the risk of posterior tracheal wall perforation
- Anatomically shaped cannula adjusts to the trachea due to “memory effect”

**LATEX
FREE**

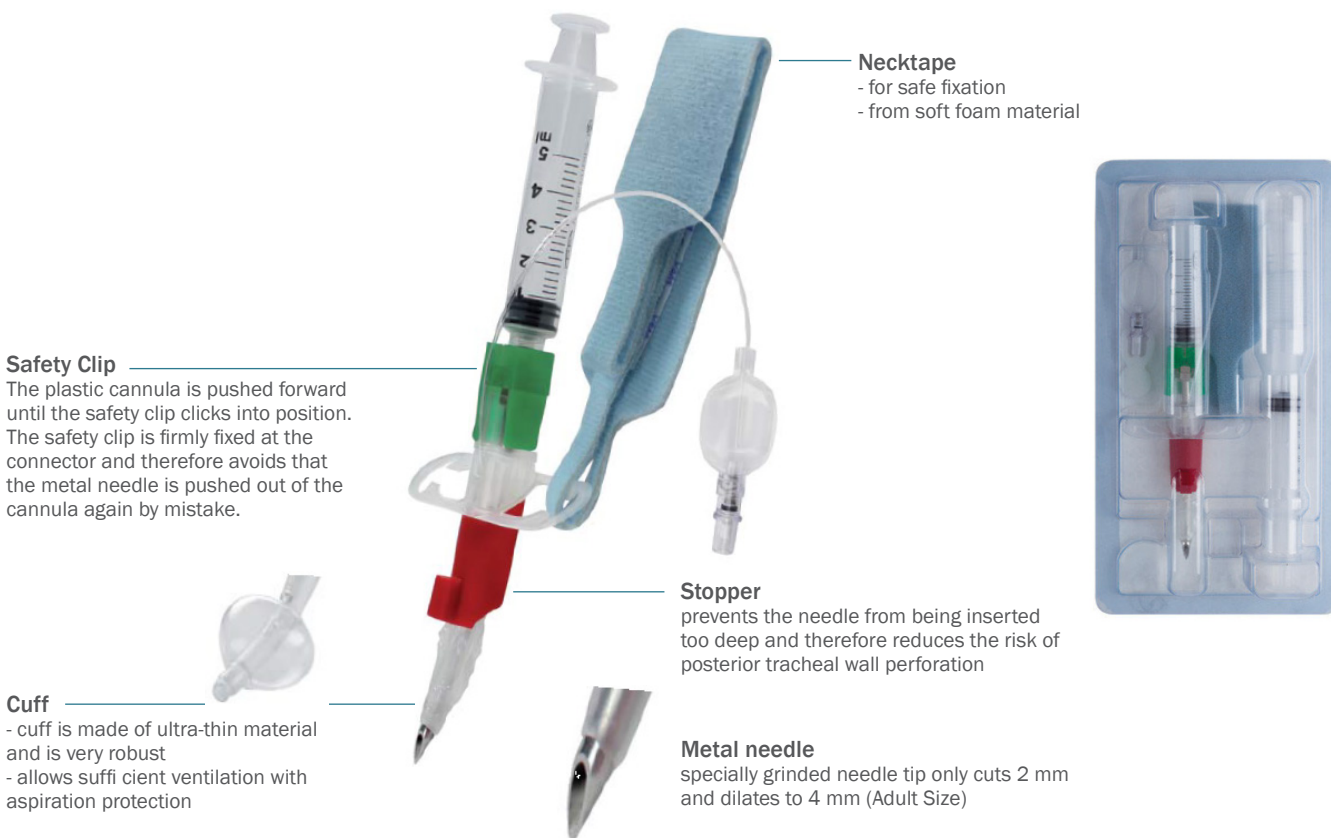

Safety Clip
The plastic cannula is pushed forward until the safety clip clicks into position. The safety clip is firmly fixed at the connector and therefore avoids that the metal needle is pushed out of the cannula again by mistake.

Cuff
- cuff is made of ultra-thin material and is very robust
- allows sufficient ventilation with aspiration protection

Necktape
- for safe fixation
- from soft foam material

Stopper
prevents the needle from being inserted too deep and therefore reduces the risk of posterior tracheal wall perforation

Metal needle
specially grinded needle tip only cuts 2 mm and dilates to 4 mm (Adult Size)

Product Code	Description	QTY
30-10-004-1	Quicktrach II with cuff Adult I.D. 4mm	BOX 1