



**STEP 1**

Insert I.V. cannula using local protocols. The skin surrounding the wound should be clean and dry.

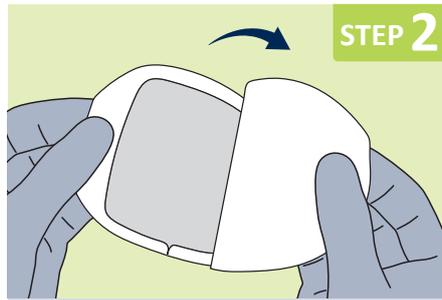


**STEP 2**

Open the sterile packaging and remove the dressing. Separate the paper backing from the AsGUARD® Clear Film Dressing.

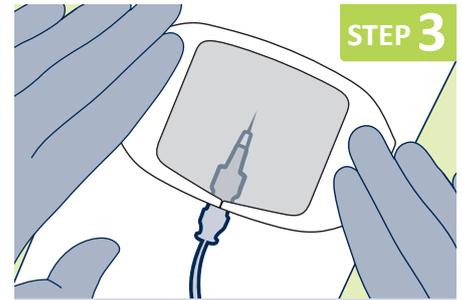

**STEP 3**

Apply the AsGUARD® Clear Film Dressing covering insertion site and I.V. cannula hub. Apply gentle pressure around the edge of dressing.

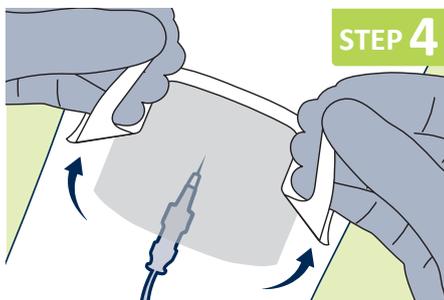

**STEP 4**

Peel the window frame away from the dressing. Smooth down the edges of the clear border to create a waterproof and secure seal. Complete and apply dating label as preferred and required by local protocol.


## INDICATIONS

AsGUARD® Clear Film Dressings are indicated for use as dressings in the management of all types of insertion sites such as peripheral IV cannulae or used as a fixation layer to provide a waterproof environment and/or to keep non-adhesive primary dressings in place.

## CONTRAINDICATIONS

- Should not be used if allergies to any of its components, is known.

## DRESSING CHANGE

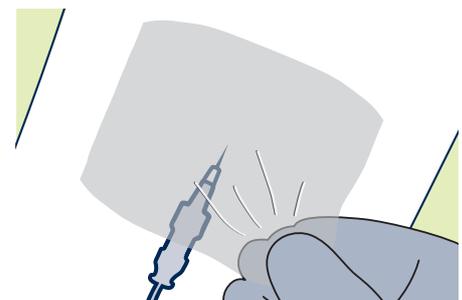

7 days is the maximum wear time for dressing.

To minimise potential damage to fragile skin, AsGUARD® Clear Film Dressing should be eased off, using a “stretch and release” technique or an adhesive remover wipe.