



**STEP 1**

Prepare site as per local protocol and ensure skin is clean and dry before application.

Separate the paper backing marked "REMOVE FIRST" to uncover adhesive side of dressing.



**STEP 2**

Apply the dressing over the device and remove final backing paper, so dressing is completely adhered to patient.



**STEP 3**

Apply gentle pressure across the surface of the dressing, ensuring adherence.



**STEP 4**

Peel clear top layer (indicated by arrows) off the dressing.



**STEP 5**

Ensure thorough adhesion by smoothing down dressing from centre to outer edges.

Apply securement tape(s) and insertion date label as preferred and required by local protocol.



**REMOVAL**

To minimise potential damage to fragile skin, AsGuard® Exceed should be eased off using the "stretch and release" technique or with an adhesive remover wipe.

## INDICATIONS

AsGuard® Exceed Dressings are intended to cover and protect IV insertion sites where High MVTR is critical e.g. PICC lines. The clear adhesive film can also be used to secure dressings over wounds and to provide waterproof protection to the wound and dressing pads, where moisture vapour transmission requires control.

## CONTRAINDICATIONS

Should not be used if allergies to any of its components is known.