


THE iPad SP1

SET-UP QUICK REFERENCE GUIDE



STANDARD PACK CONTENTS ARE:

- 1 x iPad SP1 AED
- 1 x carry case
- 2 x sets dual adult/child electrode pads
- 1 x battery pack
- 2 x AED starter kits
(CPR facial barrier, 2 dry wipes, tough cut scissors, gloves, gallant prep razor)

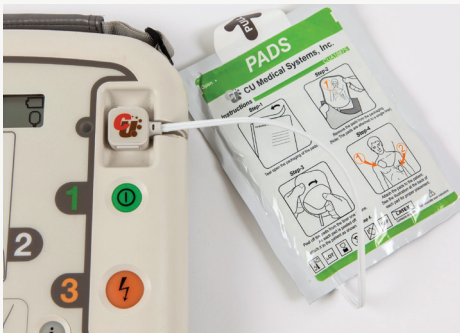

Dual adult/child electrode pads are pre-connected to the iPad SP1.



Insert the battery pack in the direction of the arrow on the label. Push it in to the iPad SP1 until you hear it click into place.

The device will start a self-check. The LCD display will display the status of the unit, the connection status of the electrode pads, the expiration of the electrode pads and battery life.

Follow the voice prompts. Press the orange shock button when asked.



Press the flashing blue i button when asked.



Place the iPAD SP1 in to the carrying bag. Gently pull the edge of the pads through the pad slot in the case.



Place one of the AED starter kits and the spare set of electrode pads in the inside pocket of the carrying bag.



The iPAD SP1 AED is now ready for use. Place in accordance with local requirements. Ideally in an easily accessible, secure place.



The status LCD is visible through the carrying bag and should be checked regularly.



Store the additional AED starter kit for replenishment if required. The additional electrode pads should remain in the inside pocket.



The local ambulance service will be informed that an iPAD AED is in place.

For technical support call

03333 444789



**TO RE-ORDER ACCESSORIES
PLEASE CALL YOUR SUPPLIER
AND QUOTE:**

- 63032 – Disposable battery
- 63124 – Dual adult/child pads
- 63040 – AED starter kit
- 63036 – Indoor alarmed cabinet
- 63138 – Wall mounting bracket

Other accessories available upon request