

Application technique*

Pack opening and handling protectors



Flat body area – “two lobes at a time” (1/3)



- Peel the protectors from the centre, without removing completely
- Centre the dressing
- Anchor some of the border on the peri-wound
- You can use your fingers to help the anchoring of the border
- Take hold of one of the partially peeled protectors

Flat body area (2/3)




- Gently peel away two lobes of protector, while laying the border down
- Do not stretch the border, pat it down

Flat body area (3/3)



- Gently shape the pad on/around the body part, without stretching it
- Gently peel away the other two lobes of protector, while laying the border down
- Do not stretch the border, pat it down

Difficult area – “one lobe at a time” (1/4)

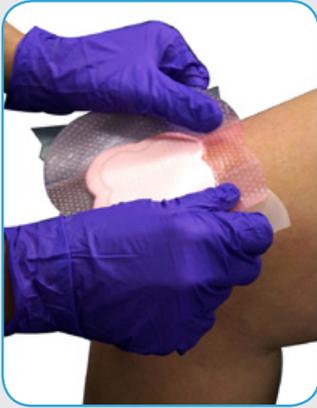


- Peel the protectors from the centre, without removing completely
- Centre the dressing
- Anchor some of the border on the peri-wound
- You can use your fingers to help the anchoring of the border
- Take hold of one of the partially peeled protectors
- Gently peel the protector off, patting the border down for one of the lobes
- The rest of the border can be left un-adhered for the moment

Difficult area (2/4)



- Gently shape the pad on/around the body part, without stretching it
- Gently peel the other protector off, patting the border down for one of the other lobes
- Do not stretch the border
- The rest of the border can be left un-adhered for the moment

Difficult area (3/4)

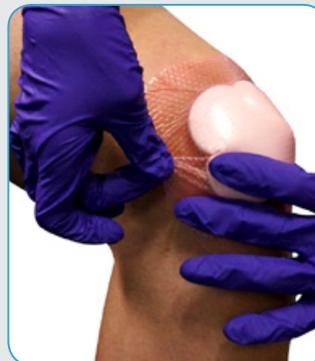

- Gently shape the pad on/around the body part, without stretching it. It can bend easily.
- Pat the border down on the skin, creating some areas of border overlap
- Let the border overlap in a wing shape, rather than with many creases (don't stretch it)
- Repeat the neat overlap on the other side
- Ensure the adhesion to skin is all the way around the pad, by smoothing the border

Difficult area (4/4) result: Movement is possible



Knee in flexion (as applied)



Knee in extension

Ordering ALLEVYN LIFE

Code	Dimensions (cm)	Carton
66801067	Small - 10.3cm x 10.3cm	10
66801068	Medium - 12.9cm x 12.9cm	10
66801069	Large - 15.4cm x 15.4cm	10
66801070	Extra Large - 21cm x 21cm	10

